

Cederroth eyewash 2 x 500 ml 725200



Product features:

Reference: 38826
EAN13: -

The Cederroth eyewash is a first-aid device designed to provide optimal rinsing in the event of eye accidents. When a chemical, such as an acid or alkaline, comes into contact with the eyes, it's crucial to start rinsing immediately to minimize damage. The Cederroth eye shower has been specially developed to ensure effective, fast and safe rinsing.

The device operates with a simple opening system and a generous flow of buffered liquid, offering a powerful neutralizing effect on corrosive splashes. The integrated eyecup protects against dust and guarantees optimum rinsing solution. This product is essential in all environments where hazardous chemicals are used, such as workshops, laboratories, or industrial sites.

Features :

- **Abundant flow:** Liquid is released in abundance for fast, effective rinsing.

- **Protected eyecup:** The eyecup is protected from dust to ensure maximum hygiene.
- **Buffered solution:** The rinsing liquid is buffered with borate, neutralizing alkaline and acidic substances more effectively.
- **Easy opening:** The shower opens automatically once released from its holder, for rapid use.
- **Flexible installation:** The shower can be flexibly positioned according to need and available space.
- **Compatibility with calcium-containing substances:** The solution is phosphate-free, making it safe to use even when exposed to products such as cement or concrete.

Advantages:

- **Fast, effective rinsing:** Cederroth Eyewash provides rapid rinsing thanks to its generous water flow, effectively neutralizing corrosive substances.
- **More effective neutralization:** Its borate-buffered liquid rapidly restores the eye's natural pH, offering superior protection compared to conventional saline solutions.
- **Maximum safety:** The protected eyecup and simple opening ensure contamination-free rinsing with total control.
- **Practical design:** Easy to install and use, it can be placed in any work environment or industrial space, and is always ready for use.
- **Long rinse mashed potato:** You can continue rinsing eyes with Cederroth Eyewash for several minutes, if necessary, for a complete clean.

Usage:

Cederroth Eyewash is an essential device for eye first aid, particularly in professional environments where corrosive chemicals are frequently handled. It is ideal for :

Industrial workshops and construction sites: Protect your employees from the risk of chemical accidents.

Laboratories: An essential piece of equipment for researchers handling potentially eye-damaging substances.

Schools and universities: Keep students safe in scientific and technical environments.

Hospitals and clinics: Provide an immediate solution for eye accidents caused by chemicals.

How to use Cederroth Eyewash:

1. **Begin rinsing immediately** upon exposure to a hazardous substance.
2. **Rinse thoroughly** with Cederroth Eyewash for at least 15 minutes.
3. **Keep the eye open** during rinsing to ensure complete cleaning of the eye surface.
4. **Continue rinsing during transport** to hospital if necessary, using additional bottles or an eye wash station.

Cederroth eyewash is an essential safety product for any situation where there is a risk of chemicals being sprayed into the eyes. Thanks to its buffered liquid and generous rinsing system, it ensures rapid, effective neutralization, reducing the risk of serious injury. Easy to install, easy to use and always ready to go, it's a reliable and indispensable first-aid solution for eye protection in professional environments.

